

THEALOZDUO®

TREHALOSE 3% | HYALURONIC ACID 0.15%

Directions for use

Wash hands well before using each of these products.

THEALOZ DUO drops can be used when wearing contact lenses. If using the gel, remove contact lenses first and wait at least 30 minutes before re-inserting.

How to use Thealoz Duo drops:



- Drop one drop in the pocket of lower eyelid
- Do not touch the eyes or eyelids with the tip of the bottle, especially when you think you may have an eye infection
- Put the cap back on after use

How to use Thealoz Duo UD:



- Take off one single-dose container from the strip
- Open the container and squeeze to release solution
- Drop one drop in the pocket of lower eyelid
- Throw away the container, even if there is some solution left

How to use Thealoz Duo Gel:



- Take off one single-dose container from the strip
- Turn upside down and tap the container
- Open the container and squeeze until you see the gel
- Drop one drop in the pocket of lower eyelid
- Throw away the container, even if there is some solution left

Dose

1 drop in each eye throughout the day as required - Gel is especially suitable for use at night time.

THEALOZDUO®

TREHALOSE 3% | HYALURONIC ACID 0.15%



"I have had dry eyes for a number of years and have tried lots of different drops with varying success. I tried Thealoz Duo recently and have been very impressed."

S. DAULMAN, NUNEATON

If you require more information please speak to your Optician

Thea Pharmaceuticals
Driving Innovation, Education and Professionalism

Thea Pharmaceuticals Limited
MED IC3 • Keele University Science & Business Park • Keele
Newcastle Under Lyme • ST5 5NP

Head Office: **0845 521 1290** • Medical Information: **0870 192 3283**
E-Mail: theasupport@theapharma.co.uk • www.theapharmaceuticals.co.uk

RE-ORDER CODE TDU022 - Printed April 2015

FOR PATIENTS WITH DRY EYE

THEALOZDUO®

TREHALOSE 3% | HYALURONIC ACID 0.15%

MEDICAL DEVICE

Sore, gritty or tired eyes?



Clinically proven to be long lasting



PROTECT, HYDRATE & LUBRICATE

Thea Pharmaceuticals
Driving Innovation, Education and Professionalism

What is Dry Eye?

Dry Eye means that your eyes may not be making enough tears, the tears you are making are of poor quality or evaporate faster than they should.

What are the symptoms?

The symptoms can include:


- Dryness
- Excessive watering
- Gritty feeling/foreign body sensations
- Pain
- Blurring of vision
- Redness
- Tired eyes
- Photophobia (sensitivity to light)
- Contact lens discomfort
- Irritation from wind/smoke



THEALOZDUO®

TREHALOSE 3% | HYALURONIC ACID 0.15%

Unique
Formulation

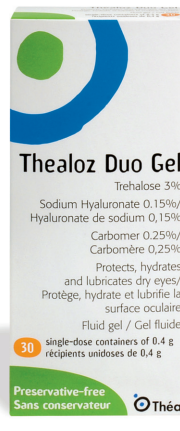

FREE
from
Preservatives



Unique formulation available in a choice of applications

Thealoz Duo Gel contains: Trehalose, Hyaluronic Acid and Carbomer

- 30 Preservative Free single unit doses
- Hypotonic gel - to restore the natural balance of tears
- Can be used alone or as an adjunct to Thealoz Duo drops
- Easy to apply, without stickiness
- Provides long lasting relief of Dry Eye symptoms as an alternative to more frequent use of drops
- Especially suitable for night time use
- Suitable for use with contact lenses (prior to wear)

Thealoz Duo 10ml contains: Trehalose and Hyaluronic Acid

- 10ml solution or 30 single unit doses
- Preservative Free
- Hypotonic solution - to restore the natural balance of tears
- Suitable for up to 3 months of use (ABAK)
- Easy to use for all patients
- Fast acting and long lasting
- Easy to squeeze bottle
- Handy size, economical, no wastage - can be used until the very last drop

